



Midwest Hearing Loops is the leading edge innovator in the design and installation of hearing loops across the Upper Midwest, including the states of Minnesota, Wisconsin, Iowa, North Dakota, and South Dakota.

We design, install and test this new technology in public settings such as Westminster Presbyterian (home of the Town Hall Forum), Hennepin County Government Center, Stillwater Public Library, and Duluth Entertainment and Convention Center

Midwest
Hearing LoopsTM
Advantage Computer Systems, Inc.

1 Hill Farm Circle
St. Paul, MN 55127
651-484-0924
midwestloops.com

*Hear more
of Life*

This Symbol

indicates the presence of a hearing loop and prompts you to turn on your T-coil Program.



If you don't have a T-coil equipped instrument, it alerts you to borrow a loop listener.

Hearing Loop

Switch hearing aid to T-coil



How to Benefit From a Hearing Loop

All you need is a hearing aid or cochlear implant with a telecoil program.

(and your current hearing instrument may already have a telecoil option)

You Can Choose Between:

- telecoil only (which will amplify only the speaker)
- telecoil + microphone (which amplifies both the speaker and the sound in the room)
- There are two separate programs: (1) T only and (2) T + mic

Note: a telecoil program is not the same as a telephone program.



Ask Your Audiologist To:

- add a telecoil program to your instrument
- demonstrate how to activate your telecoil program when you see the above sign
- practice activating your telecoil program with your Audiologist
- assure the acoustic indicator (beep or voice) is loud enough to confirm program button use

If you are unable to activate the program independently, teach a friend or family member to activate the program for you.

If you have dexterity challenges, ask for a remote control demonstration. Remote controls are available in different sizes to assist with dexterity challenges.

Midwest
Hearing LoopsTM
Advantage Computer Systems, Inc.

Hear more
of Life